



Accel INTERVention RTU & Wipes Directions for Use

Protective Barriers:

1. Disposable gloves. Gloves should be changed as required, i.e., when torn, when hands become wet inside the glove or when moving between patient rooms.
2. Household gloves can be worn, but they must be discarded when the cleaning is complete.

Dilutions:

Product is Ready to Use and does not require dilution.

Diluting of product is not recommended.

Shelf Life: 24 months from date of manufacture. Expiry date is listed on each product label.

Recommended Procedures for Cleaning and Disinfecting of Surfaces and Equipment:

1. Gather all equipment, cleaning solutions and materials required to clean the items or surfaces.
2. WASH hands and put gloves prior to cleaning. Personal protective equipment should be changed if torn or soiled.
3. Visible or gross soil present and/or blood or body fluid spills must be removed prior to cleaning and disinfection.
4. To clean and disinfect all surfaces, apply Accel INTERVention RTU or Wipes and allow surfaces to remain wet for 1 minute to achieve the Bactericidal, General Virucide and Tuberculocidal claims. To ensure that cross contamination does not occur, change cloths or disposable wipes when soiled or when moving to new surface. If using an open bucket system with the RTU liquid, ensure that solutions do not become contaminated (NO DOUBLE DIPPING).
5. Soiled rags should be placed in a bag for laundering. Disposable wipes should be disposed as regular waste in garbage bags.
6. Remove and discard gloves, WASH hands.
7. Accel INTERVention RTU is not intended for use as a holding solution for non-critical and semi-critical items.

Recommended Procedures for Cleaning and Disinfecting of Blood and Bodily Fluids:

1. WASH hands and put on gloves.
2. If the possibility of splashing exists, the worker should wear a face shield and gown. For large spills, overalls, gowns or aprons as well as boots or protective shoe covers should be worn. Personal protective equipment should be changed if torn or soiled and always removed before leaving the location of the spill.
3. Apply Accel INTERVention RTU to spill – wait 30 seconds.
4. Blot up the blood with disposable towels. Dispose of paper towel in plastic-lined waste receptacle.
5. Spray or wipe surface with Accel INTERVention RTU or Wipes – wait 1 minute. Wipe dry with disposable paper towel. Discard paper towel as above.
6. Remove gloves and dispose in plastic-lined waste receptacle.
7. WASH hands.